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English

How do you effectively protect your hearing?

- Make a habit of it to consequently wear your hearing protectors in areas with excessive noise levels. If you do not do this, the average attenuation will decrease considerably.
- Never wear the hearing protectors without filter.
- Depending on your personal noise load, a filter of 15, 19, or 22 dB attenuation was selected. The filters themselves can be exchanged.
- Have your hearing protectors checked regularly (annually) for their functionality. Checking the seal of the hearing protectors is of great importance for its effectiveness.

How do you apply your hearing protectors in your ear?

- Take the hearing protectors from the supplied case with clean hands. The hearing protectors are provided with a filter module, marked with a red or a blue dot. Red is for your right ear, blue is for your left ear.
- For your right ear it is best to take the hearing protector in your right hand at the little grip of the filter module, so that the red dot points to the right and is positioned in front.
- Pull the auricle of your right ear slightly up with your left hand whilst you apply the channel tip of your hearing protection into the auditory duct, and subsequently turn it a quarter turn backwards. Subsequently, you can still push the hearing protector for a moment, should it not be fully in place.
- Take the left hearing protector into your left hand and apply it in the same manner.

How do you maintain your hearing protectors?

- Correct maintenance can considerably extend the life-span of your hearing protection.
- Carefully keep your hearing protectors in the case supplied.
 - Do not expose the hearing protectors to extreme temperatures.
 - Clean the hearing protectors after use with the supplied cloth.
 - If the auditory duct is clogged with earwax, you best use the supplied earwax bud to remove this earwax. NEVER use another object to remove the earwax.
 - If you remove the filter, the hearing protectors can be cleaned with lukewarm water. The MEP 2G must be completely dry before you may once again place the filter. Prevent contact between the filter and water in any case. It is advisable to regularly clean the hearing protectors. Hygienic tablets are efficient for quick cleaning. These are available from your supplier.

Replacing the filter

Indications for replacing the filter are, among other things:

- Discolouration
- Crack or visible defect
- Lesser performance

1 pair of filters is supplied as standard. These are already placed in the hearing protection. One can also reorder a pair of filters with different attenuation if desired. You can replace this yourself in the following way:

- Take the hearing protector out of the case, and hold it firmly at the channel tip.
- Hold the filter module with your other hand and make a tipping move so that it comes off.
- You can simply click the new filter straight into the provided space.
- Always keep your spare filters at a dry and cool location.
- The filters are marked with a blue, white or black ring at the bottom of the module. This stands for 15, 19, or 22 dB respectively.

Technical data

The MEP 2G complies with all criteria for the CE quality mark EN 352-2: 2002 (DIN EN 352-2: 2003), according to the test report 091102-1. (PZT, Notified Body, GmbH, D-26389 Wilhelmshaven).

Attenuation values of the various filters:

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Mf = average attenuation | Sf = standard deviation

APVf = assumed protection value

SNR-values

Filter blue	SNR = 15 dB	H = 14 dB	M = 13 dB	L = 13 dB
Filter white	SNR = 19 dB	H = 16 dB	M = 18 dB	L = 19 dB
Filter black	SNR = 22 dB	H = 19 dB	M = 21 dB	L = 22 dB

Contact your distributor for more information about this product. You can contact us via info@variphone.com or +32 (0) 11 668 120.

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